INSTRUCTIONS FOR USE

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- | | | |
|---|---|---|
| (EN) Medical compression stockings | (NO) Medisinske kompresjonsstrømper | (HU) Kompressziós gyógyharisnya |
| (DE) Medizinische Kompressionsstrümpfe | (DA) Medicinsk kompressionsstrømpe | (RO) Ciorapi compresivi medicinali |
| (FR) Bas médicaux de compression | (FI) Lääkinnälliset kompressiosukat | (CS) Zdravotnické kompresivní punčochy |
| (ES) Medias de compresión médica | (IS) Teygjusokkar til lækninga | (SL) Medicinske kompresijske nogavice |
| (PT) Meias de compressão medicinais | (ET) Meditsiinilised kompressioonsukad | (SK) Zdravotné kompresné pančuchy |
| (IT) Calze compressive medicali | (LT) Medicininės kompresinės kojinės | (RU) Медицинские компрессионные чулки |
| (NL) Medische compressiekousen | (LV) Medicīniskās kompresijas zeķes | (TR) Tıbbi kompresyon çorapları |
| (SV) Medicinska kompressionsstrumpor | (PL) Medyczne pończosznicze wyroby kompresyjne | (AR) جوارب طبية ضاغطة |
| | | (HE) גרבי לחץ רפואיים |

FIBER CONTENT

Product ¹ Produit	Polyamide ² Polyamide (Nylon)	Elastane (Spandex) ³ Elasthanne (Spandex)	Silicone ⁴ Silicone	Polyester ⁵ Polyester	Other ⁶ Autre
Active Wear 15-20 / 20-30	20 %	9 %	0 %	60 %	11 % Cotton ⁷
Active Wear 30-40	20 %	12 %	0 %	58 %	10 % Cotton ⁷
Style SoftFit 15-20	53 %	12 %	2 %	0 %	16 % Pima Cotton ⁸ + 17 % MicroModal ⁹
Style SoftFit 20-30	53 %	17 %	2 %	0 %	14 % Pima Cotton ⁸ + 14 % MicroModal ⁹
Style SoftFit 30-40	45 %	26 %	2 %	0 %	14 % Pima Cotton ⁸ + 13 % MicroModal ⁹
ForMen 15-20 KH	76 %	24 %	0 %	0 %	–
ForMen 15-20 TH	73 %	20 %	7 %	0 %	–
ForMen 20-30 KH OT	71 %	29 %	0 %	0 %	–
ForMen 20-30 TH	71 %	25 %	4 %	0 %	–
ForMen 20-30 / 30-40 KH CT	73 %	27 %	0 %	0 %	–
ForMen 30-40 KH OT	71 %	29 %	0 %	0 %	–
ForMen 30-40 TH	71 %	25 %	4 %	0 %	–
ForMen Casual 15-20	46 %	14 %	0 %	35 %	5 % Wool ¹⁰
ForMen Casual 20-30	47 %	16 %	0 %	33 %	4 % Wool ¹⁰
ForMen Casual 30-40	48 %	16 %	0 %	31 %	5 % Wool ¹⁰
ForMen Ambition 15-20	61 %	23 %	0 %	1 %	15 % Cotton ⁷
ForMen Ambition 20-30	49 %	28 %	0 %	1 %	22 % Cotton ⁷
ForMen Ambition 30-40	55 %	29 %	0 %	1 %	15 % Cotton ⁷
ForMen Ambition SoftFit 15-20	56 %	24 %	3 %	1 %	16 % Cotton ⁷
ForMen Ambition SoftFit 20-30	52 %	28 %	2 %	0 %	18 % Cotton ⁷
ForMen Ambition SoftFit 30-40	48 %	33 %	3 %	1 %	15 % Cotton ⁷

KH¹¹
Knee high
Mi-bas

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1. Product description

JOBST® medical compression stockings incorporate gradient compression for the management of venous and lymphatic conditions.

2. Intended purpose

JOBST® medical compression products are designed to exert external, physical compression. Every JOBST® medical compression stocking has a specific compression level designed to help manage specific venous and lymphatic conditions, for thrombosis prophylaxis and as a postoperative treatment.

The required compression class and style must be determined by your healthcare professional. The compression level is printed on the package label.

Indications

JOBST® Medical compression stockings are used for the management of the following indications:

Compression level 15-20 mmHg*:

- Tired, aching legs
- Minor ankle, leg and foot swelling
- Minor varicosis
- Thrombosis prophylaxis in mobile patients

Compression level 20-30 mmHg*:

- Tired, aching legs
- Moderate to severe varicosis
- Post vein surgery
- Post sclerotherapy
- Moderate tendency of edema
- For adjunct management of superficial venous thrombosis
- Helps prevent recurrence of ulcerations

Compression level 30-40 mmHg*:

- Severe varicose veins
- Edema
- Active patients with lymphedema in ISL stage I-III with no or minimal shape distortion (not applicable for Style)
- Post vein surgery
- Post sclerotherapy
- Venous ulcers
- Chronic venous insufficiency (CVI)
- Manages manifestations of post-thrombotic syndrome (PTS)
- Helps prevention of post-thrombotic syndrome (PTS)

*The mean compression for an average ankle size. Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

3. Contraindications

JOBST® medical compression stockings are contraindicated (should NOT be worn) if you have one or more of the following medical conditions:

- Severe peripheral arterial occlusive disease
- Uncontrolled / (decompensated) heart failure
- Septic phlebitis
- Phlegmasia coerulea dolens
- Untreated and / or advancing skin infection (e.g. erysipelas) involving the area where the stockings will be applied
- Open wounds not covered with an appropriate dressing
- Incompatibility to material

4. Warnings and precautions

Medical compression stockings should only be used after consultation with your healthcare professional and checked regularly by him / her on the appropriateness of your therapy.

Your healthcare professional decides whether compression in general can be applied to patients suffering from polyarthritis (unspecified), complex regional pain syndrome (CRPS, Morbus Sudeck), malignant lymphedema, and medical conditions in which increased venous and lymphatic return is not desired.

Consult your healthcare professional if you have experienced or develop: controlled / compensated heart failure, mild to moderate peripheral arterial occlusive disease, peripheral neuropathy, severely compromised skin sensibility and impaired sensitivity of the limb, as well as skin infection or weeping dermatoses.

If you notice discomfort, pain, numbness, changes in skin color, constrictions, pressure marks, appearance of new wounds or swelling whilst wearing, stop using the product and contact your healthcare professional.

Patient measurements should be taken by a trained person to determine correct product fit. Incorrectly measured and badly fitting products may cause constriction leading to e.g. skin necrosis, pressure ulcers, and pressure-related peripheral nerve damage during use.

When re-ordering compression stockings after 6 months we recommend remeasuring the patient as the limb circumference might have changed.

Appropriate hygiene and skin care should be applied.

Appropriate footwear must be worn when walking to help prevent falls.

Not recommended to wear during bed rest.

Caution is required if you are confined to bed or unable to walk.

5. Directions for use

To get the best medical result from your JOBST® medical compression stockings, wear them regularly and consistently.

Ideally you should put on your compression stockings in the morning, immediately after getting up. You may experience some difficulty putting on the stockings in the beginning but this will become easier after a number of days.

When applying the compression stockings, please do not pull on the edge of the stockings. Follow the recommendations below.

Turn the compression stocking inside out all the way to the heel. Pull apart the folded-in foot part using both hands and pull over the foot all the way to the heel.

Gradually pull the compression stocking over instep and heel.

Then gradually pull the compression stocking upward by turning the stocking to a right-side out position piece-by-piece.

Distribute the stocking evenly by massaging with both hands until the stocking fits without forming creases. It is particularly helpful to wear rubber gloves for this step.

For a waist high garment continue working the garment up to the middle of your thigh. Stand up. With both hands, grab a handful of fabric and pull it up to your waist.

Adjust your support so that it is snug at your midsection. This will keep it from sliding down. Pull the midsection to its proper height.

Slip your hands, palm out, into the back and lift the fabric up and away from your skin. Let it fall back to fit snugly over the buttocks.

Special notes for donning/doffing:

Long fingernails and jewelry may damage the stockings when putting them on.

For easier application use rubber gloves or a donning aid.

Do not bring the compression stockings into contact with greasy ointments, body lotions or similar substances unless recommended by JOBST®.

Loose threads or loops should not be cut off, since this may cause holes or runs to form in the stockings.

6. Cleansing and care

We recommend to wash the JOBST® compression stocking on a daily basis, or at least every other day.

For this purpose, you may use a mild detergent and a low spin cycle setting. Do not use laundry additives such as fabric softener, optical brightener or stain remover.

Rinse your compression stockings well.

Roll up the wet compression stockings in a towel after washing and press out excess moisture. Do not wring.

Do not dry the compression stockings on a radiator or in direct sunlight.

Machine washable in warm water.

Do not bleach.

Flat drying or tumble dry - low heat.

Do not dry clean.

Do not iron.

Note: a laundry net provides additional protection for the knitted fabric.

Silicone band care:

When needed, hand wash silicone band to remove excess creams, lotions, hair and other foreign material.

7. Storage

Due to wear or slackening of the material the stockings are intended for a wearing period of six months. This assumes correct handling (e.g. product care, donning and doffing).

Please protect your compression stockings from direct exposure to sunlight, heat and moisture. We recommend storage at room temperature.

8. Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical Inc. (BSN-JOBST® GmbH in the European Union) and the competent authority of your state.

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Medical
device
Dispositif
médical

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Date of
Manufacture
Date de
fabrication

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16013-90019-00

	Artwork ID: JBR18883.03	Review Loop: 01	Printing Colours:  Pantone 281 C
Project Title: C2847_JOBST_MLW_REY		Date of Review Loop: 28.07.2023 KW_KLND	
Contact ArtworkManagementMedical@essity.com		Replaces Artwork ID: JBR18883.02	
Contact Address: BSN medical GmbH Schützenstraße 1–3 22761 Hamburg Germany		Technical Drawing for print ID	
Languages: EN, DE, FR, ES, PT, IT, NL, SV, NO, DA, FI, IS, ET, LT, LV, PL, HU, RO, CS, SL, SK, RU, TR, AR, HE		Note LOT and expiry-date symbol will be part of variable coding	

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BSN-JOBST GmbH
BSN medical (Aust.) Pty Ltd
Essity UK Ltd • Southfields Road